

ProMOTION®

EV1



USER MANUAL

Version 2.1

www.promotion.fitness

CONTENTS

ProMOTION® EV1 Wellness Device	3
Product Specifications	3
Classification	4
Warnings	6
Contraindications	7
Device Overview	9
Using Your ProMOTION® EV1 device	15
Menu Overview	27
ProMOTION® EV1 App	32
Cleaning, Maintenance & Storage	35
Troubleshooting	37

ProMOTION® EV1 WELLNESS DEVICE

Thank you for choosing the ProMOTION® EV1 wellness device. To ensure you have the best experience possible, please be sure to read the entire user manual prior to use of the product.

The ProMOTION® EV1 is a therapeutic product designed to assist a user's lifestyle, combining thermal and compression therapies for key muscle sets and skeletal joints, for use by individuals undertaking sports and general exercise.

The ProMOTION® EV1 integrates cooling, heating, and compression therapy in a unique, fully portable system. Compression and temperature are easily programmed and adjusted at the touch of a button for specific programs and user needs, with no requirement for water, ice, or heavy equipment. Note that the ProMOTION® EV1 is not intended to be used for medical purposes.

Please feel free to let us know what you think of our products, we'd love to hear from you.

customerservices@swellaway.com

PRODUCT SPECIFICATIONS

Pressure level: Ranges from 20 mmHg up to 75 mmHg

AC POWER INPUT: 100-240V, 50/60Hz, 0.8A.

DC POWER INPUT: 18V, 36W

Operating Temperature: 6 – 40 °C.

Recommended Ambient Operating Temperature: 18 – 25 °C

Recommended Operating Humidity: Non-condensing

Swellaway Ltd.

Great Oak Farm Offices, Mag Lane, Lymm, Cheshire, WA13 0TF, UK

www.promotion.fitness

www.promotion.fitness/resources



CLASSIFICATION

This equipment has been tested and complies within the limits for household and similar electrical devices in EN 60335-1:2012/A11:2014 and EN 60335-2-32:2003/A2:2015 Low Voltage Directive (LVD 2014/35/EU). These standards specify the general requirements and conditions to test the domestic safety of electrical household appliances including defining the classification and marking of electrical equipment, protection against live parts, heating, leakage currents, electric strength at operating temperatures, moisture resistance, resistance to heat and fire, stability and mechanical hazards, internal wiring and supply connections.

Electromagnetic Compatibility

This Therapy Unit has been approved against the following standards for Electromagnetic Compatibility (EMC):

- EN 61000-6-2:2019;
- EN 61000-6-3:2007+A1:2011;
- ETSI EN 301 489-1 V2.2.3;
- ETSI EN 301 489-17 V3.2.4

Radio Emissions

This Therapy Unit has been approved against the following standards for Radio Emissions:

- ETSI EN 300 328 V2.2.2
- EN 62311:2008*

Safety Testing

This Therapy Unit has been approved against the following standards for Safety testing:

- EN 60335-1:2012/A11:2014
- EN 60335-2-32:2003/A2:2015

The Therapy Unit's battery pack has been approved against the following standards for Safety testing:

- UN 38.3
- IEC 62133-2:2017*
- JIS C62133-2:2020 to PSE Standard








CLASSIFICATION

Ingress Protection

Note that this is a general-purpose product with minimal protection against water. Always keep the device dry.

Flammable Anaesthetics or Oxygen

The product is not suitable for use in an oxygen enriched environment or in the presence of flammable anaesthetics.

	Powered by direct current.
	Attention: Consult accompanying documents before use.
	Attention: Consult accompanying documents for cautionary information before use.
	Storage temperature limits (-20°C to 35°C).
	WEEE symbol.
	Fragile parts. Handle with care.
	Keep dry.

WARNINGS

Important Instructions. Please retain for future use.

A copy of these instructions is available on our website.

- ⚠ Monitor the therapy area before, after and during therapy. If any painful, pressure, burning, loss of feeling, itching, increased swelling or pain, or skin changes such as an allergic reaction, blisters, increased redness or discolouration occur, cease use and remove the product immediately and contact Swellaway Ltd. Improper placement or prolonged use of the device could result in damage.
- ⚠ Do not cover the air vents of the *ProMOTION*® EV1 device. This may cause the device to overheat. If during use the cooling fan stops, or you suspect any other component failure, cease use and remove the product immediately and contact Swellaway Ltd.
- ⚠ Monitor the *ProMOTION*® EV1 device and its components during use. The produce contains electronics. If any burning smell and/or excessive noise is experienced, cease use, remove the product immediately, remove the battery from the device and contact Swellaway Ltd.
- ⚠ If during use the *ProMOTION*® EV1 screen or app shows a warning or error message, cease use and remove the product immediately and contact Swellaway Ltd.
- ⚠ *ProMOTION*® EV1 compression wraps are not sterile; do not place directly against open wounds, sores, rashes, infections or stitches.
- ⚠ The *ProMOTION*® EV1 device contains electronics and should be disposed of in line with regional disposal regulations. Do not dispose of the device's electrical components in normal waste.
- ⚠ Use of the *ProMOTION*® EV1 device in extreme temperature, pressure and humidity environments may cause reduced device functionality. Do not use the device on an aircraft.
- ⚠ Handle the *ProMOTION*® EV1 device with care. Never operate the device with damaged or modified components or if the unit is otherwise not fully operational.
- ⚠ If you suspect battery leakage, stop usage immediately. Batteries should be disposed of carefully in line with local authority regulations.
- ⚠ Do not use any cables, charging adapters or accessories other than those provided by with the *ProMOTION*® EV1.
- ⚠ The *ProMOTION*® EV1 compression wraps may be available in multiple configurations, please make sure to use them in accordance with the varying user guides.
- ⚠ Do not use the device whilst asleep.
- ⚠ Do not use in damp or wet environments. Exposure to rain may damage the device.

- ⚠ The appliance is not to be used by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, been given supervision or instruction.
- ⚠ Children should be supervised to prevent them playing with the device.
- ⚠ Examine the appliance frequently. Return to the supplier if it has been misused.
- ⚠ Not intended for medical use.
- ⚠ Not for use by children.
- ⚠ Persons insensitive to heat must be careful when using the appliance.

CONTRAINDICATIONS

- ⚠ The ProMOTION® EV1 is not intended or designed for medical purposes
- ⚠ **Caution:** Compression therapy using the ProMOTION® EV1 device should not be used by persons:
 - Who are in the acute stages of inflammatory phlebitis in the affected region;
 - Who have any current clinical signs suggestive of deep vein thrombosis in the affected region;
 - Who have significant arteriosclerosis or other vascular ischemic disease in the affected region.
 - Who have any significant risk factors or current clinical signs of embolism (e.g. pulmonary embolus, cerebral infraction, atrial fibrillation, endocarditis, myocardial infarction, or arteromatous embolic plaque);
 - Who have a condition in which increased venous or lymphatic return is not desired in the affected extremity (e.g., carcinoma);
 - Who have decompensated hypertonia in the affected region;
- ⚠ **Caution:** Cooling therapy using the ProMOTION® EV1 device should not be used by persons:
 - Who have reduced skin sensitivity to avoid the potential risk of frostbite. (e.g. locally anaesthetised areas from nerve root compression syndromes, diabetic users, users with circulatory disorders, open wounds or skin conditions, users with iron deficiency anaemia, and/or poor kidney function);
 - Who have sensitivity or aggravation to cold (e.g. Raynaud's syndrome, thyroid conditions, severe cardiovascular disease, severe hypertension and those whose joint symptoms are aggravated by use of cold);
 - With areas where nerves have become more superficial (e.g. ulnar nerve at the elbow and the peroneal nerve at the head of the fibular, cold-induced neuropathy in the axillary nerve and the lateral femoral cutaneous neuropathy);
 - Who have known hematological dyscrasias which affect thrombosis (e.g., paroxysmal cold hemoglobinuria, cryoglobulinemia, sickle-cell disease, serum cold agglutinins).

CONTRAINDICATIONS

⚠ Caution: Compression therapy with the *ProMOTION*® EV1 device should be used only under the supervision of a licensed healthcare practitioner on users:

- Who have a wound in the affected region (the wound must be dressed prior to use of the *ProMOTION*® EV1 device);
- Who have an acute, unstable (untreated) fracture in the affected region;
- Who are children under 18 years old or users who have cognitive disabilities or communication barriers, whether temporary due to medication or permanent;
- Who have a cardiac insufficiency or congestive heart failure;
- Who have a localized skin condition (e.g. dermatitis, vein ligation, gangrene, skin graft) in the affected region;
- Who have erysipelas or other active infection in the affected region.
- Who have Raynaud's disease or cold hypersensitivity;
- Who have hypertension or extreme low blood pressure;
- Who have diabetes;
- Who have compromised local circulation or neurologic impairment (including paralysis or localised compromise due to multiple surgical procedures) in the affected region;
- Who have rheumatoid arthritis in the affected region;
- Who have known and uncontrolled peptic ulcer (if the affected region is the abdominal region) since cold therapy applied to the abdomen can cause increased gastrointestinal motility and gastric acid secretion.

⚠ Caution: Cooling Therapy with the *ProMOTION*® EV1 device should be used only under the supervision of a licensed healthcare practitioner on users:

Please note: There may be some instances where your healthcare practitioner may decide that cold and compression therapy are appropriate for individuals who have some of the above conditions. This decision is always a clinical judgment call and must always be made by a suitably qualified practitioner.

DEVICE OVERVIEW

The *ProMOTION*® EV1 pack includes the following items; a user manual, a Therapy unit, a Pump unit, two batteries, an AC charging adaptor, a SM/MD Compression wrap, a MD/LG Compression wrap and a universal strap.



- A - Therapy Unit
- B - Compression Pump Unit
- C - Battery Pack (x2)
- D - Small/Medium Compression Wrap
- E - Medium/Large Compression Wrap
- F - Universal Sock with 2 Straps (small & large)
- G - AC Charging Adaptor
- H - Portable Carry Case

Not shown:

***ProMOTION*® EV1 User Manual**

Bladder bung (1 per Compression wrap)

Please note that the device date and time are programmed by the App. In order to ensure that the device correctly logs the date and time of a program, use the App to control the device when it is first used. This procedure will program the device with the date and time held on the App.

DEVICE OVERVIEW

Therapy Unit


The *ProMOTION*® EV1 Therapy Unit features a multifunction interactive keypad and LCD display. The keypad can be used to power the unit on, select a program mode, start or stop a program and to power the unit down.

The LCD display indicates; power status, program selection, program status and error messages.

The Therapy Unit also contains a Pump Connection Port located adjacent to the LCD display. This is used to connect to the Pump unit.




DEVICE OVERVIEW

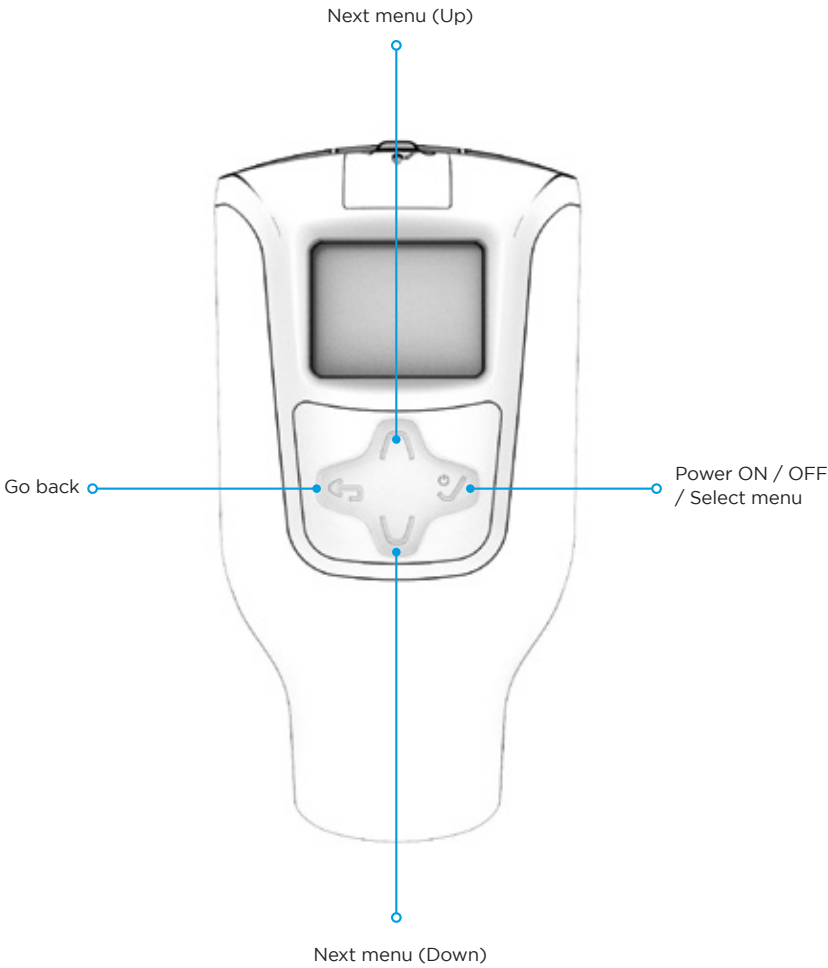

DEVICE OVERVIEW

Pump Unit

The ProMOTION® EV1 Pump Unit features a USB connector and clip for connection to the Therapy Unit. It also features a bayonet fitting for connection to the Compression Wrap.


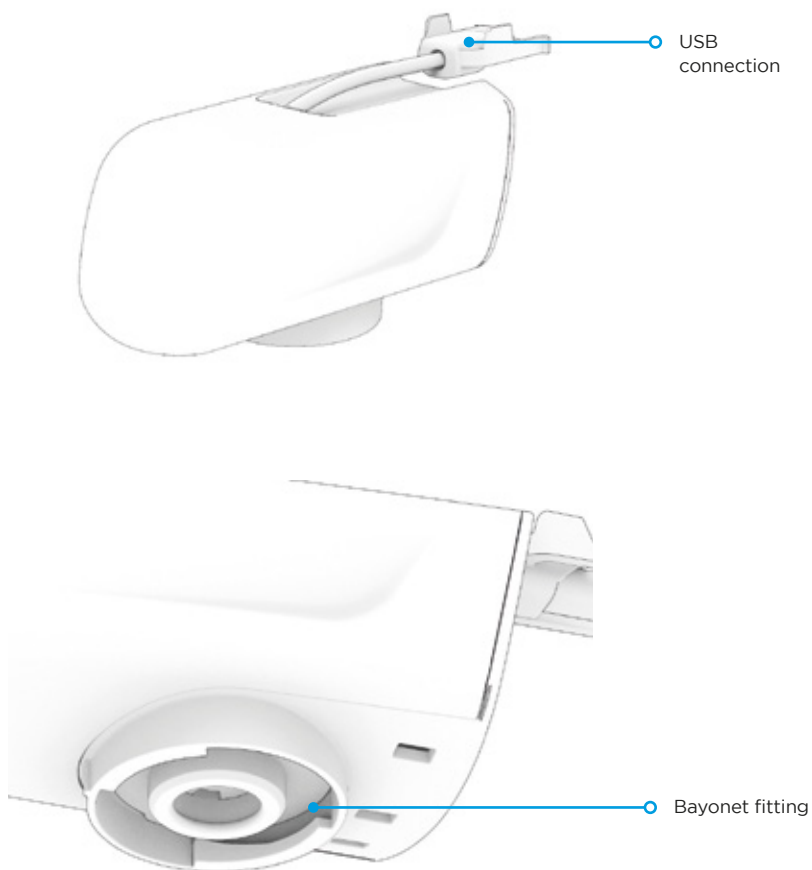


DEVICE OVERVIEW

Battery

The ProMOTION® EV1 battery features a battery level indicator and button, charging port, and eject tab. Depressing the indicator button will result in the level of charge being indicated.

- 1 LED = 0 - 25% charge
- 2 LED = 25 - 60% charge
- 3 LED = 60 - 90% charge
- 4 LED = 90 - 100% charge

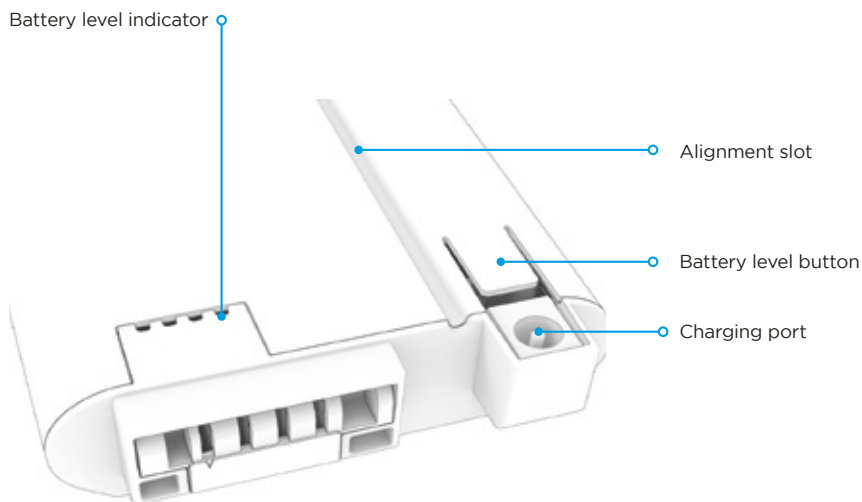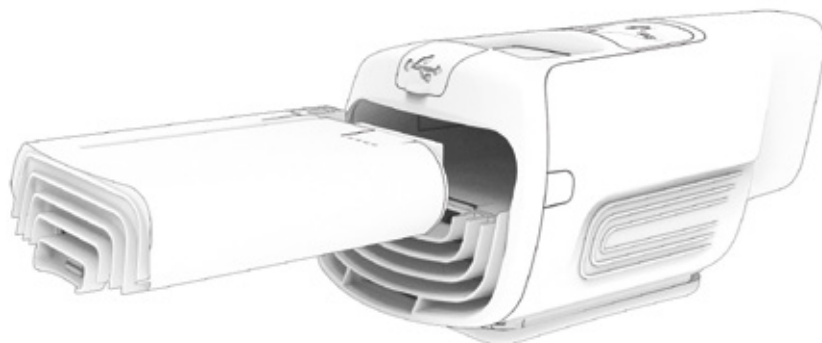


DEVICE OVERVIEW

Compression wrap

The *ProMOTION*® EV1 Compression wraps feature a cradle for docking the Therapy Unit and Pump Unit. The wraps also feature an adjustment array and handle for adjusting the wrap to suit.

Note that two sizes of wrap are supplied with the *ProMOTION*® EV1 system. Small / Medium (S/M) and Medium / Large (M/L).

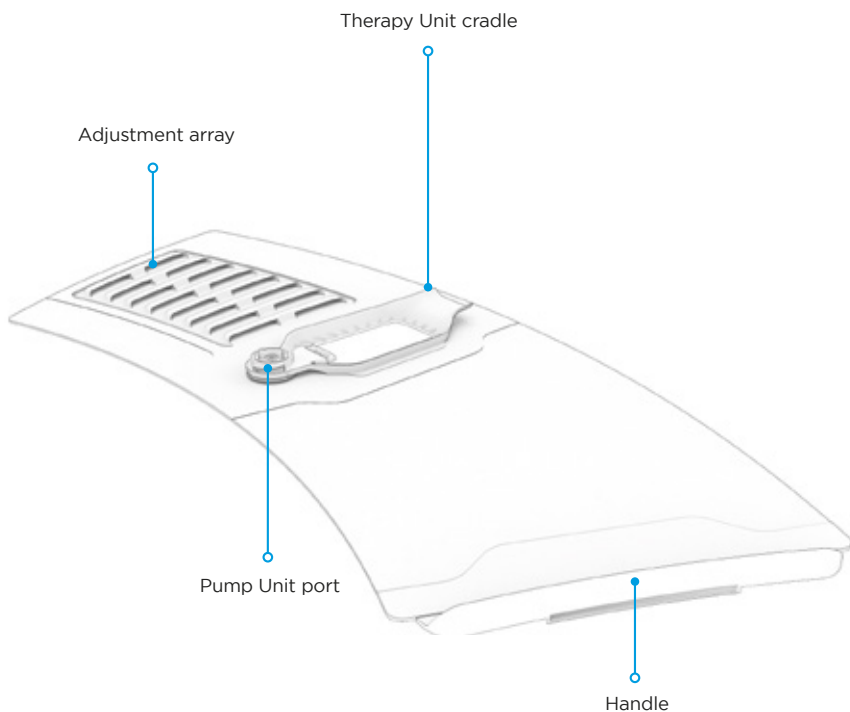


USING YOUR ProMOTION® EV1 DEVICE

Battery charging and usage


The battery should be fully charged before use to ensure maximum performance. Follow the procedure below before using the device.

- ① If the battery is installed in the Therapy Unit, lift the eject latch on the battery. The battery will spring out.

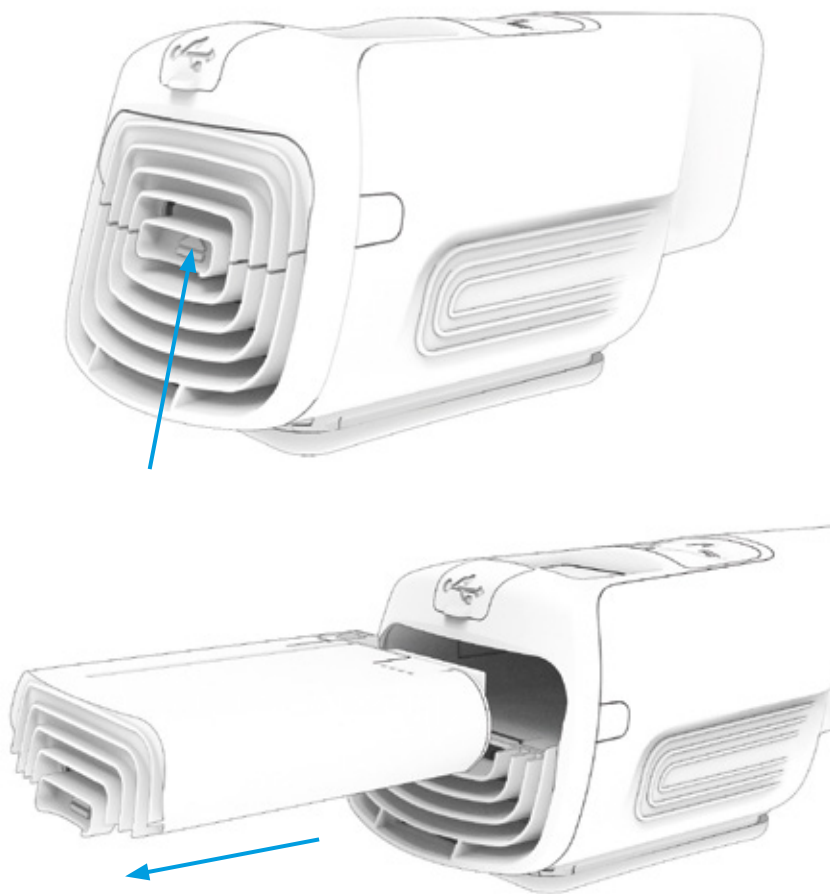

USING YOUR ProMOTION® EV1 DEVICE

- ② Connect the charger to the battery charging port, then connect the charger to a mains power supply and turn the supply on.
- ③ The battery level indicator will light up and sequentially pulse to indicate that charging is in progress. The percentage of charge is indicated by how many LEDs pulse.
 - 1 LED = 0 - 25% charge
 - 2 LED = 25 - 60% charge
 - 3 LED = 60 - 90% charge
 - 4 LED = 90 - 100% charge
- ④ Unplug the charging cable from the battery charging port once the battery is charged.

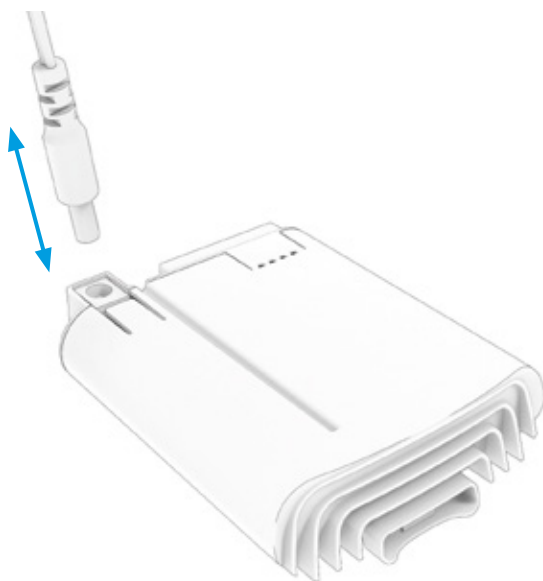


Precautions

- ⚠ Only use supplied charging cable.
- ⚠ The battery should never be covered during the charging cycle.
- ⚠ It is recommended that battery charging is ceased once the battery has reached a fully charged state.


USING YOUR ProMOTION® EV1 DEVICE

Battery Installation

- ① Orientate the battery with the Therapy Unit. A slot on the battery will align with the corresponding feature inside the Therapy Unit.

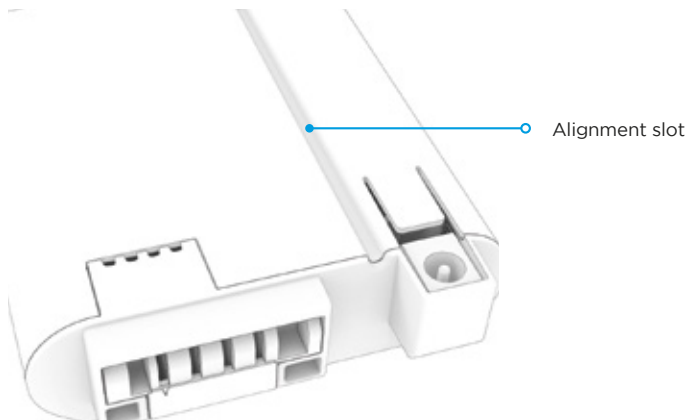

- ② Push the battery into the Therapy Unit until the battery eject tab clicks, securing the battery in place.

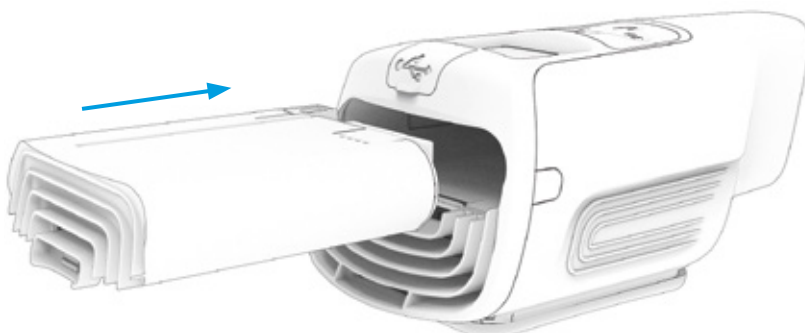

Precautions


- ⚠ Do not force the battery into the Therapy Unit. Check the battery is correctly aligned and nothing is obstructing the battery slot.

USING YOUR ProMOTION® EV1 DEVICE

Installing Therapy Unit

The Therapy Unit can be used on its own for the application of heating or cooling. If you wish to use compression therapy too, the Therapy Unit should be paired with the Pump Unit and Compression wrap. To do this:

- ① Align the Therapy Unit with the cradle.
- ② Position the slot in the front of the Therapy Unit against the plastic spring, and push the Therapy Unit forwards (see image below).
- ③ Lower the Therapy Unit down into the cradle.
Allow the spring to push back, locking the Therapy Unit in place.

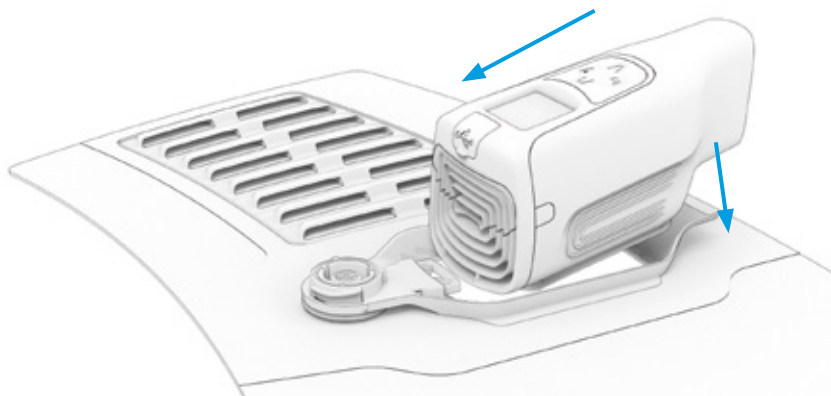

Precautions


- ⚠ Check the Therapy Unit is secure by pulling it up and away from the cradle.
- ⚠ Do not obstruct the air inlet or exhaust at either end of the Therapy Unit.

USING YOUR ProMOTION® EV1 DEVICE


Installing Pump Unit

The Pump Unit attaches to the Compression wrap via a simple bayonet fitting. To attach it:

- ① Align the Pump Unit with the port on the cradle. Align the pips on each component (see image below).

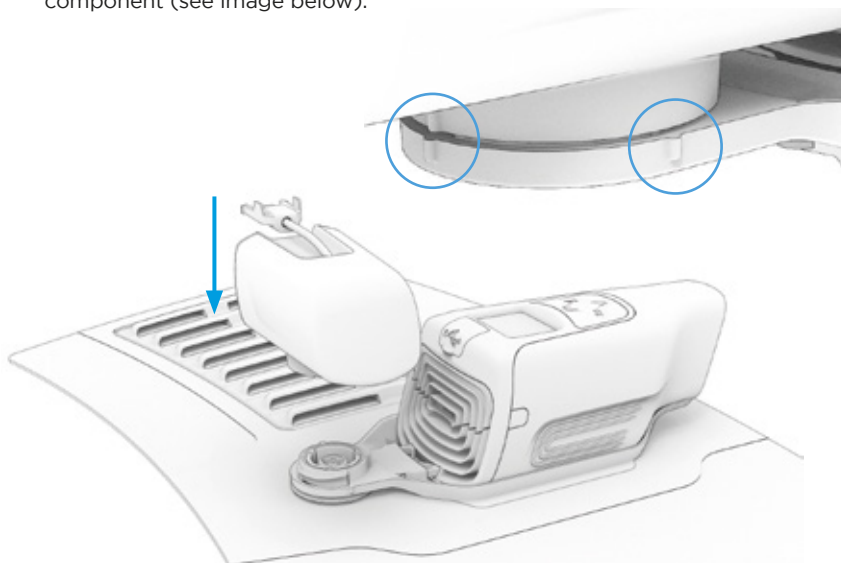

- ② Gently support the compression wrap under the bayonet
- ③ Start to rotate the Pump Unit clockwise. The bayonet will begin to engage.

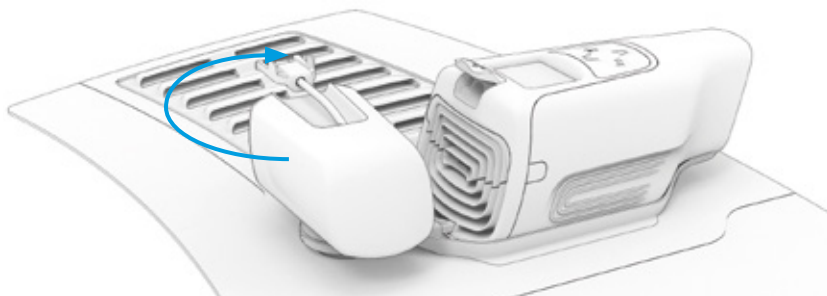


continues overleaf


USING YOUR ProMOTION® EV1 DEVICE

Installing Pump Unit

- ③ Continue to rotate the Pump Unit, until it is in line with the Therapy Unit.

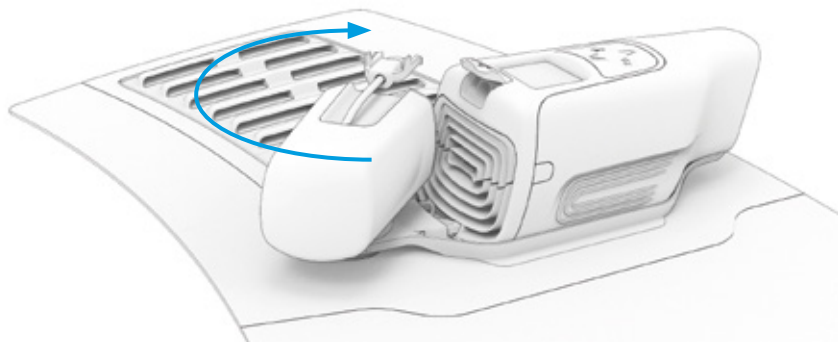

- ④ Push the Pump unit cable and clip into the USB port on the Therapy Unit until it clicks into place. The small grips on the USB clip should be orientated upwards.

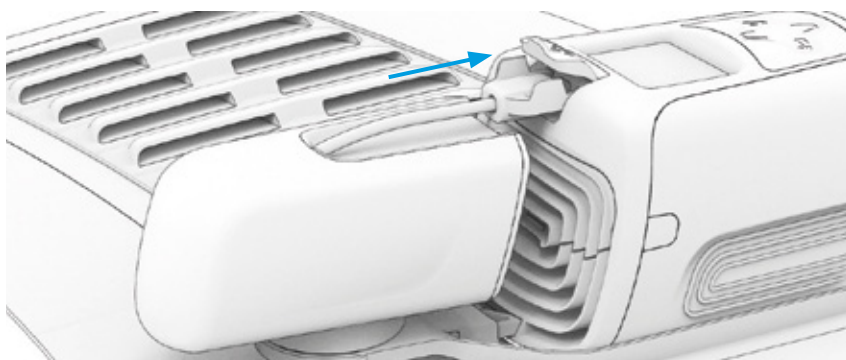


Precautions


- ⚠ Check that the Pump Unit fits the bayonet port correctly before use. The Pump Unit should be 'parallel' with both the Therapy Unit and the cradle.

USING YOUR ProMOTION® EV1 DEVICE

Fitting Compression wrap

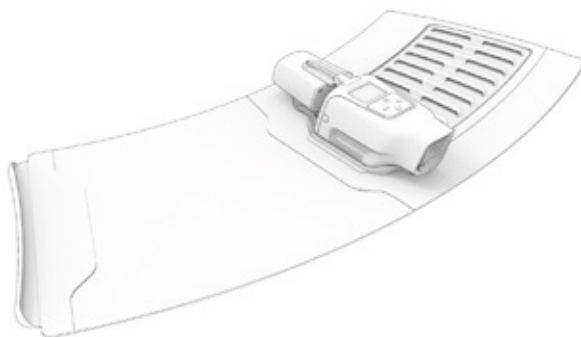

- ① Carefully locate the assembly by placing the Therapy Unit over the injury site.
- ② Wrap the Compression wrap comfortably around the limb - It should be secure enough not to slip, but not tight.
- ③ Position the wrap handle into the adjustment array, securing the wrap assembly in place.

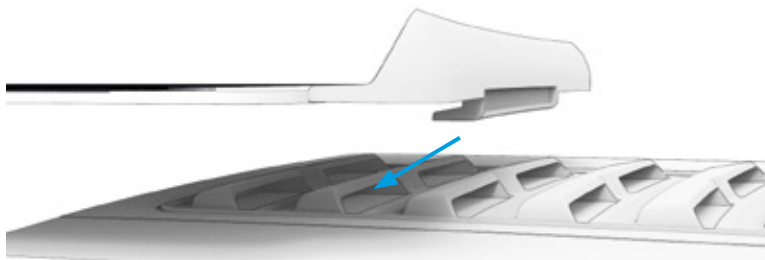

Precautions

- ⚠ Do not over tighten the Compression wrap around the limb. The wrap will secure itself when inflated.

USING YOUR ProMOTION® EV1 DEVICE

Fitting the universal sock



- ① Place the smaller end of the EV1 Therapy Unit into the Universal Sock.
- ② Using both the small tab located at the top of the sock and the bottom plastic front lip of the sock to guide the Therapy Unit into the sock until you are able to click the plastic front lip of the sock behind the Therapy Unit plate. You may have to guide the bottom end of the sock over the fan. Ensure that the window of the sock is aligned with the Therapy Unit screen.
- ③ Straps are available in varied lengths depending on your need. After selecting the most suitable strap, if not already attached, connect your chosen strap by passing the VELCRO® hook tab up through the plastic loop located on the side of the sock (hooks facing outward) and secure back onto the strap.
- ④ The Universal Sock and Therapy Unit can now be secured to the body as required. Hold the EV1 therapy unit in position and pass the strap around the body. You can then hook in the quick release buckle to the stitched loop located on the side of the sock. Start with the strap loose and whilst holding the EV1 therapy unit in position, use VELCRO® hook tab to adjust the strap tension to suit. For the long strap, adjust the buckle to the general length, but the final tensioning should be done with the VELCRO® hook tab, pulling against the elasticated side loop.


USING YOUR ProMOTION® EV1 DEVICE

Powering up and using the device

The ProMOTION® EV1 device is now ready for use. Use the keypad to turn the device on, navigate the menus, and turn the device off again.

- ① Press the 'Select' button to power the Therapy Unit on.

Key pad

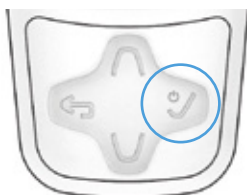


Display




- ② Use the keypad to navigate the menus. Press the 'Up' & 'Down' buttons to cycle through the menus or adjust the levels. Use the 'Select' and 'Go back' buttons to make a selection or return to the previous menu. Temperature, compression and duration can be adjusted and selected.

Key pad

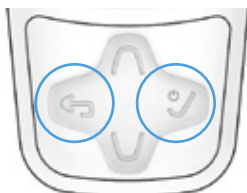

Display

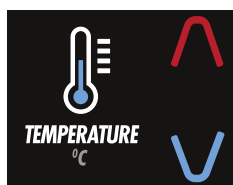

USING YOUR ProMOTION® EV1 DEVICE


Powering up and using the device

- ③ Once you have selected a program, press 'Select' to begin. A timer will appear in the display counting down the time to program completion, while the keypad will light up blue or red depending on the chosen temperature. To pause or stop a program, hold the 'Go back' button until the on-screen padlock displays as open. Then press the button again to stop the program.


Key pad



Display

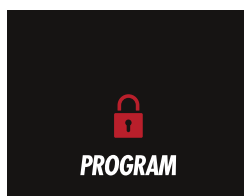


- ④ The Compression wrap will start to deflate before the end of the program. Once the program is complete, the device will revert to standby mode and the wrap can be removed. Once the wrap has deflated, gently pull the handle free of the sizing array.

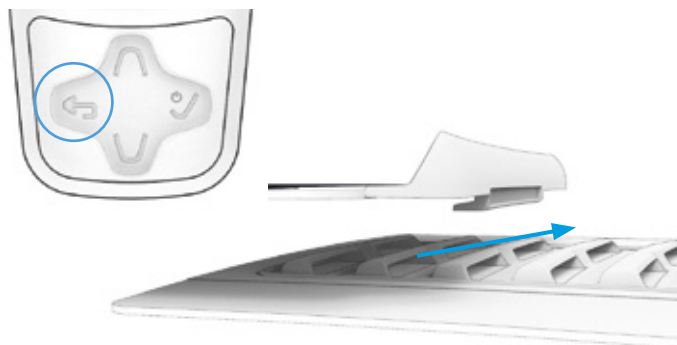

USING YOUR ProMOTION® EV1 DEVICE

Powering down

- ① Press and hold the 'Select' button to turn the Therapy Unit off.

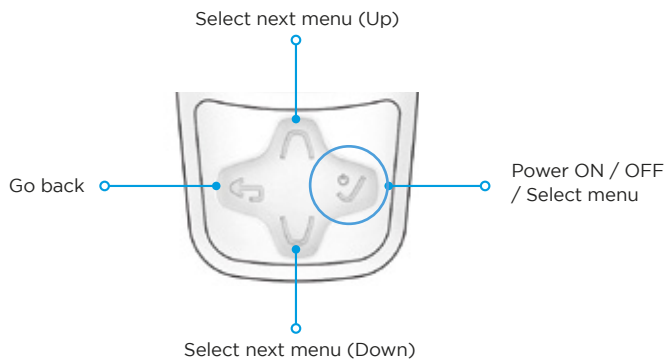


USING YOUR ProMOTION® EV1 DEVICE

Removing the pump and therapy unit


Precautions

- ⚠ Do not attempt to remove the wrap while it is inflated. Wait a few seconds after the device has switched off to allow the air pressure to drop.

- ① Squeeze and gently pull the Pump connection clip to remove it. Then rotate the pump anticlockwise to remove it.

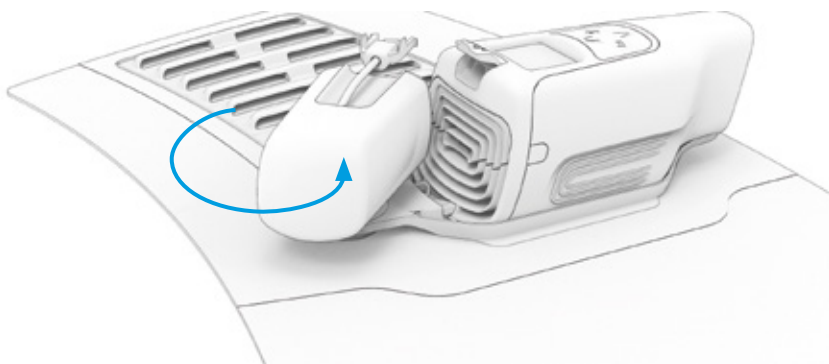


- ② Push the Therapy Unit forwards while lifting it out of its cradle.

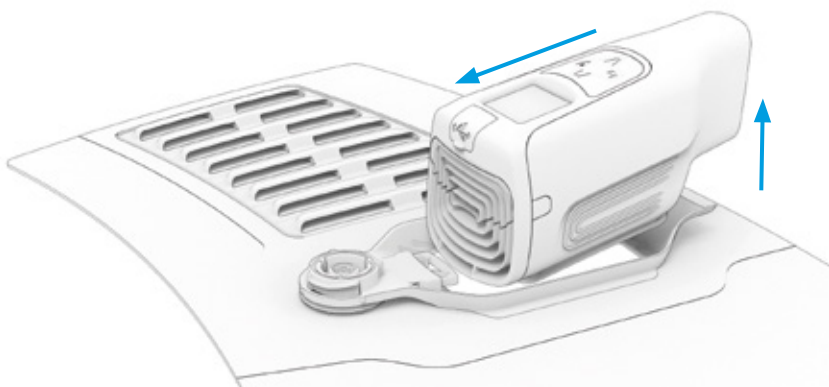

MENU OVERVIEW

The *ProMOTION*® EV1 menu system enables users to create and repeat new therapy programs, and access pre-designed routines.

Use the keypad to navigate to each menu. Pressing the up or down keys will adjust the setting. Once you are happy with your choice, press 'Select' - the device will retain your input.

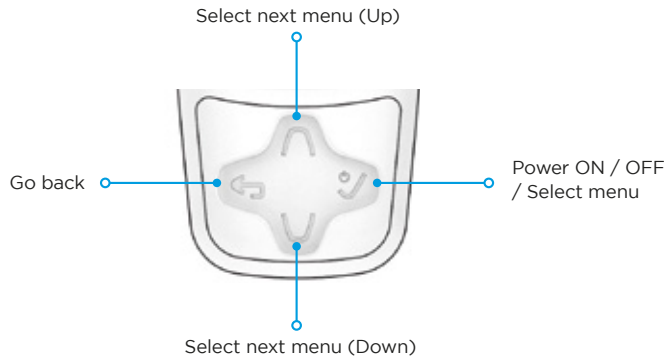

After powering up, the Therapy unit will display a series of menus enabling you to operate and program the device. 'New', 'History' and 'Programs' give access to new, old and pre-programmed routines.



New

Selecting 'New' will create a new program, where temperature, program duration and compression levels can be amended to suit.



History


Selecting 'History' allows you to access previously completed programs.

MENU OVERVIEW




Programs

Selecting 'Programs' allows you to access pre-defined programs, such as those created by a Physiotherapist or other Healthcare professional.

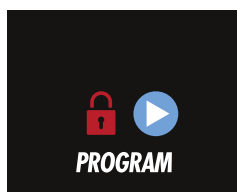

Padlock (closed) - Device in use

During a program, this screen is displayed. Press and hold 'Go back' to unlock the screen. Press 'Go back' again to stop the program.

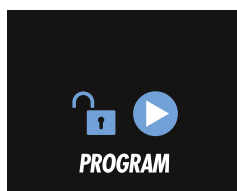


Padlock (open) - Device unlocked

This menu displays when the device is unlocked. Press 'Go back' again to stop the program.



Battery low

The battery does not have enough charge to complete a program. Please remove it for charging.

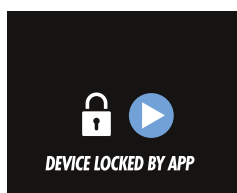


Locked

The device is under control of the app.

MENU OVERVIEW

New - Selecting 'New' will create a new program, where temperature, program duration and compression levels can be amended to suit.


- ① Use the keypad to navigate to each menu (Temperature, Duration, Compression).
- ② Press 'Up' or 'Down' to adjust the setting.
- ③ Press 'Select' to proceed to the next menu.
- ④ Once you are happy with your choices, press 'Select' to proceed to the summary page. Press 'Select' to initiate the program.

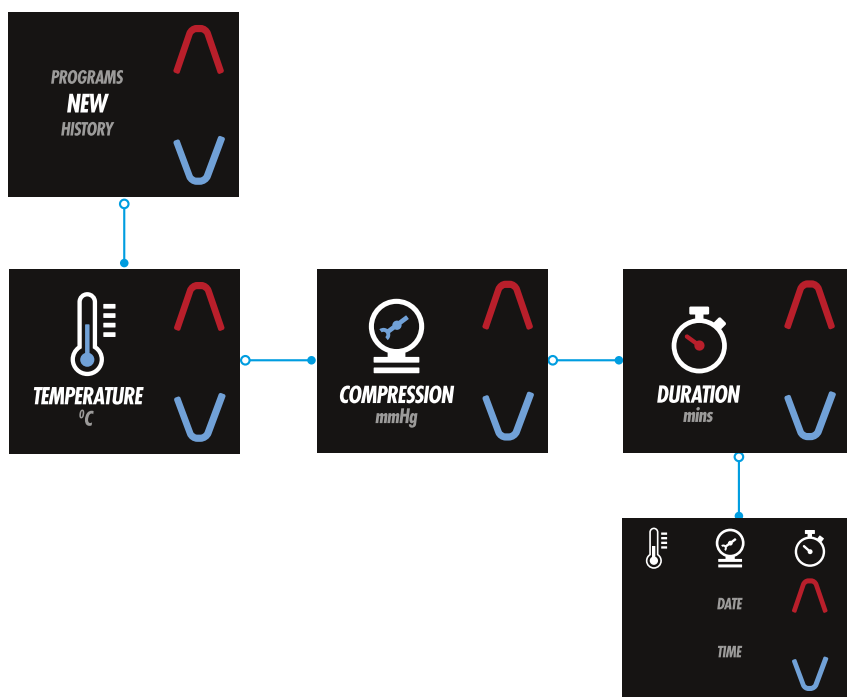

MENU OVERVIEW


History - Selecting 'History' gives access to the last 10 previously completed programs. Note that the *ProMOTION*® EV1 App gives further access - up to 100 programs.

- ① Pressing 'Up' or 'Down' allows the user to view the program details.
- ② Press 'Select' proceed to the summary screen.
- ③ Press 'Select' to initiate the program.

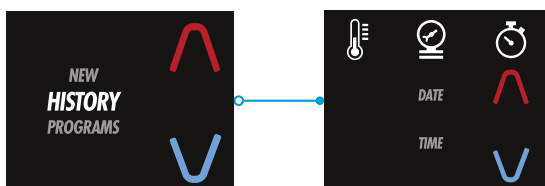

Programs - The 'Programs' function gives access to pre-programmed therapies. 10 slots are available, the first 3 of which are default programs created by *ProMOTION*® (see next page). Programs created in the *ProMOTION*® EV1 App can be saved in the remaining slots.

- ① Select 'Programs', then press 'Up' or 'Down' to choose a program.
- ② Press 'Select' to view a program's details. Contrast therapies will display each cycle as a number e.g. 01/05, with the details of each cycle displayed to the left of the screen.
- ③ Press 'Select' proceed to the summary screen.
- ④ Press 'Select' to initiate the program.

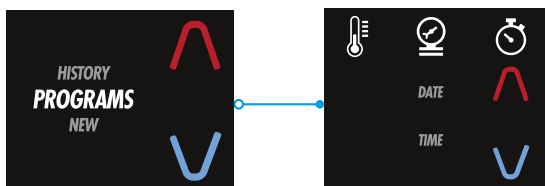

MENU OVERVIEW

Default Programs


- ❶ Cold: 6°C, 75mmHg for 20 minutes.
- ❷ Hot: 40°C, 30mmHg for 20 minutes.
- ❸ Contrast: Cycles between 6°C @ 75mmHg and 40°C @ 30mmHg for 20 minutes.

Precautions

- ⚠ Please do not remove or separate components from each other while the device is in use. Doing so may cause system error or damage.
- ⚠ If the Pump connection cable or Pump Unit are disconnected at any stage, simply reconnect it. The Pump Unit will take a few moments to recalibrate before continuing the program.
- ⚠ The Therapy Unit will not recall incomplete or interrupted programmes. If the battery is removed during a program, the device will reset from startup.
- ⚠ Please note that using the system in an environment with a high or low ambient pressure or temperature may affect the ability for the unit to provide adequate cooling, heating or compression.
- ⚠ Excessive movement during a program is not advised, as this can alter pressure in the compression system.
- ⚠ Failure to follow the set-up instructions properly may result in inadequate therapy. Ensure all components are present before use.
- ⚠ Power the Therapy Unit down before removing it from the cradle.
- ⚠ It is recommended that the *ProMOTION®* EV1 device is disassembled and placed back into protective casing for storage transport.

ProMOTION® EV1 APP

ProMOTION® Fitness app can be found on the App store or Google play store. Search for 'Promotion.fitness' - Look for the ProMOTION® logo.

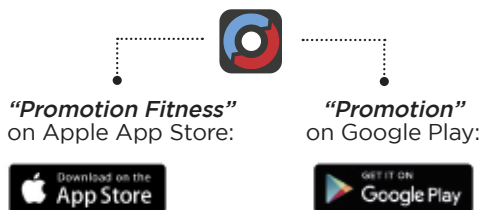

The ProMOTION® EV1 App allows the user to create and store customised therapy programs, targeting specific temperature zones or compression levels, and also giving the user the ability to create and use contrast therapy cycles.

In addition, the app can be used to create essential user information and notes, view therapy history, save history logs to PC & Mac, and share files on Android and iOS.

Please note that the device date and time are programmed by the App. In order to ensure that the device correctly logs the date and time of a program, use the App to control the device when it is first used. This procedure will program the device with the date and time held on the App.

To access this functionality, you must first connect the Therapy Unit to your app device.

Bluetooth connectivity

- ① Turn the Therapy Unit on, and initiate the ProMOTION® EV1 App.
- ② In the app, search for your device.
- ③ Select your device from list and wait until it connects - your device screen will be locked out once connected.

Using the App

From the Home screen, the app will display a series of options:

- Select 'Default Program' to select a predefined program;
- Select 'New Program' to define your own program;
- Select 'Program History' to access previously completed programs.

ProMOTION® EV1 APP

Default Programs

The ProMOTION® EV1 device and app allocates ten program slots. Three of these are default programs created by ProMOTION® EV1, the rest are available for users to create their own programs for use at a later date. Programs created here are automatically saved when you initiate them.

Programs 1-3

- ① Select a program slot - the app will display the parameters of the program.
- ② Press the 'Select' button to start your program.

Programs 4-10

- ① Select a program slot.
- ② Select 'Fixed' or 'Contrast' from 'Program Type'*.
- ③ Name your program.
- ④ Create the parameters of the program - Select a menu and use the up and down keys to adjust the setting.
- ⑤ Press 'Select' to start your program.

*Note that Contrast Therapy provides multiple cycles with the opportunity to tailor each cycle depending on your requirements.

ProMOTION® EV1 APP

New Programs

The New Program option in the *ProMOTION®* EV1 App allows the users to create a program from scratch. These can be later saved using the app.

- ① Select 'New program'.
- ② Select 'Fixed' or 'Contrast' from 'Program Type'.
- ③ Create the parameters of the program.
- ④ Press 'Select' to start your program.

Program History

The *ProMOTION®* EV1 App allows you to access the last 100 program logs used by the device. To access the list of programs.

- ① Select 'Program History'.
- ② Scroll through the history logs. They are listed chronologically, from newest to oldest.
- ③ Tap a program to open it up and review its parameters.
- ④ Press 'Select' to start the program.

History logs can be downloaded from the app in the form of a CSV file. To do this select the download button from the program summary page

Please note: App functionality may change over time

CLEANING, MAINTENANCE & STORAGE

Therapy Unit & Pump Unit


The exterior of the device can be cleaned with a soft, damp cloth. Do not use any solvent-based cleaners or abrasive materials. Examine the device frequently.

⚠ Caution: The *ProMOTION*® EV1 is not a waterproof device. Do not apply a direct stream of any liquid onto the device, submerge it, or allow any liquid to pool on the device.

Compression Wrap & Strap

The fabric parts of the *ProMOTION*® EV1 can be cleaned with cold water, using a soft, damp cloth and mild detergent or antibacterial soap.

⚠ Do not immerse the wrap in liquid. Use the supplied bung to prevent water from entering the internal bladder (see image below). Always remove the Therapy and Pump Units from the Compression wrap before cleaning. If needed, the plastic parts of the wrap may be cleaned by wiping down with commercial non-bleach cleaning wipes or hand washed using a very small amount of mild detergent or antibacterial soap. **DO NOT MACHINE WASH.** Hang to dry.

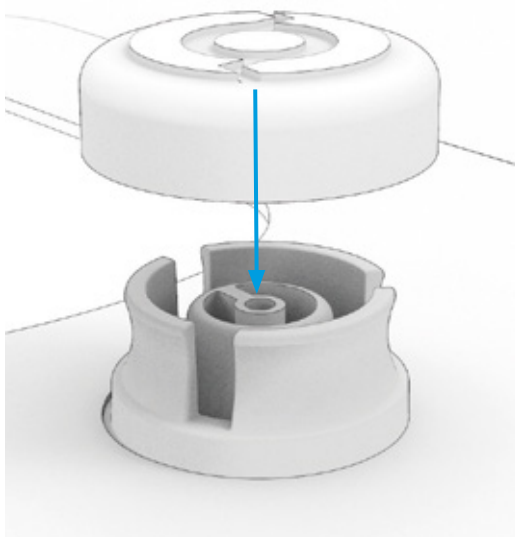

CLEANING, MAINTENANCE & STORAGE

Maintenance

It is recommended that users inspect and clean the device weekly, although this is dependent on usage. No product maintenance beyond cleaning should be performed by the user. If your device requires maintenance, please contact ProMOTION® EV1 customer services.

Storage

- Recommended storage temperature conditions: 1°C - 35°C (34° - 95°F)
- Relative humidity: 10% - 95% non-condensing.


Battery storage information

- Recommended battery storage temperature conditions: 1°C - 35°C (34° - 95°F)
- Storing the battery at temperatures outside of the recommended range may negatively affect the life of the battery.
- Store the battery with charge for optimum life. Do not store the battery in a discharged condition.

Precautions

- It is recommended that the ProMOTION® EV1 device is used at Tropospheric Pressure levels (1.075 Bar). The device should not be used on an aircraft.
- Do not leave the device in direct sunlight. The UV light may damage or discolour the device.

TROUBLESHOOTING

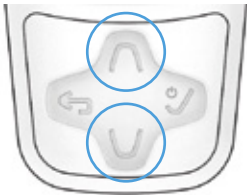Error description	Possible cause
Therapy Unit will not turn on or function as desired	<ul style="list-style-type: none"> • Battery charge may be insufficient - please recharge • Check that battery is correctly inserted into the Therapy Unit • Check for foreign object contamination in the battery connections • Battery may be damaged or expired - please contact <i>ProMOTION®</i> Customer services • Ambient Temperature & Humidity may be outside recommended range - wait until within range • If the problem persists, please contact <i>ProMOTION®</i> EV1 Customer Services
Therapy Unit will not achieve desired temperature	<ul style="list-style-type: none"> • Battery charge may be insufficient - please recharge • Ambient Temperature & Humidity may be outside recommended range - wait until within range • Battery may be damaged or expired - please contact <i>ProMOTION®</i> EV1 Customer services
Pump Unit or fan does not operate as desired	<ul style="list-style-type: none"> • Is the Pump Unit connected properly? - Check that the Pump connection cable is securely plugged into the Therapy Unit • Check that the Pump Unit is correctly mounted to the cradle. The Pump should fit parallel with the bayonet • Pump Unit or fan may be damaged - please contact <i>ProMOTION®</i> EV1 Customer services
Compression Wrap does not inflate as expected	<ul style="list-style-type: none"> • Check that the wrap is not damaged or punctured - please contact <i>ProMOTION®</i> Customer services if you require a new wrap • Ensure the bladder valve is correctly located into the cradle, and that the Pump Unit is correctly attached to the cradle • Check that the wrap is not tightly stretched around the limb prior to inflation - the wrap should only be tight enough to not slip • If the problem persists, please contact <i>ProMOTION®</i> EV1 Customer Services
Battery does not charge / no lights displayed during charging	<ul style="list-style-type: none"> • Check that the charger is correctly inserted into the battery pack • If the problem persists, please contact <i>ProMOTION®</i> EV1 Customer Services
<div style="display: flex; justify-content: space-between;"> <div style="width: 45%;"> <p style="text-align: center;">Factory reset</p>  </div> <div style="width: 50%;"> <p>A Factory reset will delete the:</p> <ul style="list-style-type: none"> • Custom programs • User info • User notes • Program history and logs. <p>From the home menu, scroll until 'New' appears on the screen. Press and hold 'up' button for 15 seconds, then down then 'down' button to confirm when prompted</p> </div> </div>	

Customer Services

01925 751 027

customerservices@swellaway.com



Email: customerservices@swellaway.com
www.promotion.fitness/resources



ProMOTION[®]

EV1

www.promotion.fitness